




Directions THC Forensic Test:

1. Open one special buffer solution bottle and take a full scoop of the spatula with the suspected plant material (approx. 50 milligram (0,05 gram) or more).
2. Shake the bottle for 30 seconds and let it stand for at least 10 minutes.
3. Shake the bottle for another 30 seconds and tap the bottom of the bottle a few times on a hard surface.
4. Snap off the tip of the bottle and add 4 drops to the sample well of the cassette with intervals of 5 seconds.
5. Wait a few seconds to 5 minutes for the line(s) to appear. Do not read after 10 minutes. The test tested positive if there is a line at the C but no line at the T. If there is a C and a T line then the test is negative even if the T line is faint. If there is no C line then the test is invalid.

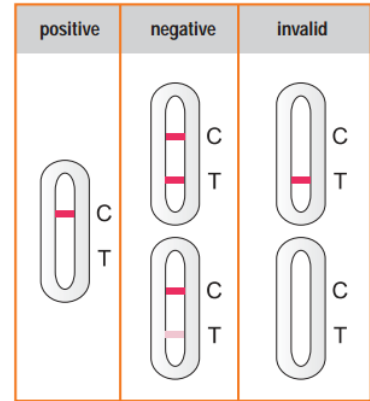

MMC INTERNATIONAL B.V. – www.narcotictests.com – info@mmcinter.com – Phone: +31-76-5711140